# Aqua+ for Kanso<sup>®</sup> 2 Sound Processor

User Guide

EN DA FI NO SV LT LV ET

CPI150

*Hear now. And always*

  
Cochlear<sup>®</sup>

# Contents

<b>EN</b>	Aqua+ for Kanso® 2 Sound Processor User Guide	2
<b>DA</b>	Brugervejledning til Aqua+ til Kanso® 2 lydprocessor	19
<b>FI</b>	Kanso® 2 -puheprosessorin Aqua+- vesisuojan käyttäjän opas	36
<b>NO</b>	Brukerveiledning for Aqua+ Kanso® 2-lydprocessor	53
<b>SV</b>	Bruksanvisning till Aqua+ för Kanso® 2-ljudprocessor	70
<b>LT</b>	„Aqua+ for Kanso® 2“ garso procesoriaus naudotojo vadovas	87
<b>LV</b>	Aqua+ Kanso® 2 palīgierīces skaņas procesora lietotāja rokasgrāmata	104
<b>ET</b>	Heliprotsessori Kanso® 2 tarviku Aqua+ kasutusjuhend	121

## Contents

About this guide .....	3
Aqua+ for Kanso® 2 description .....	4
Aqua+ for Kanso 2 instructions.....	5
Safety Line instructions.....	10
Care and storage.....	12
Cautions .....	13
Warnings .....	14
Other information.....	15
Labelling symbols .....	16
Serious incidents.....	18

## About this guide

The **Cochlear™ Aqua+ for Kanso® 2** is a re-usable sealable cover that keeps your Cochlear Kanso 2 Sound Processor (model number: CP1150) dry when used in or around water (e.g. when swimming or in the bath). It is worn on your implant with your sound processor sealed inside.

This Aqua+ is intended for use by people who use the Kanso 2 Sound Processor. It should only be used by people who can either remove the sound processor themselves if it's causing discomfort or who can indicate any discomfort to their parent or caregiver.

The **Nucleus® Safety Line** is a retention option to help prevent accidental loss of your Aqua+ by securing it to your clothing with a snap-lock clip.

While using Aqua+ you can also use the following, providing they are protected from water and kept dry:


- your smartphone (with the Cochlear Nucleus Smart App)
- Cochlear CR310 Remote Control
- Cochlear wireless accessories.

Testing shows Aqua+ can be:


- used at depths of up to 3 metres for up to 2 hours.
- re-used up to 50 times, but this may vary according to how you use it.

There have been no reported undesirable side effects from using Aqua+.

## Aqua+ for Kanso® 2 description

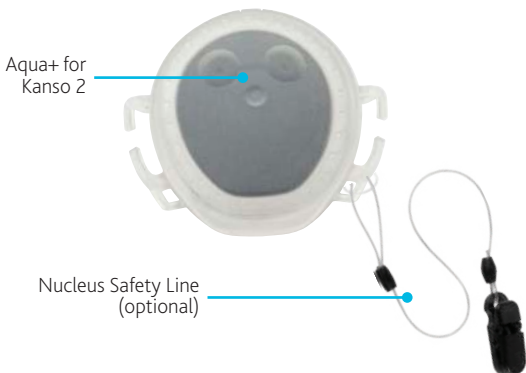### Aqua+ cover
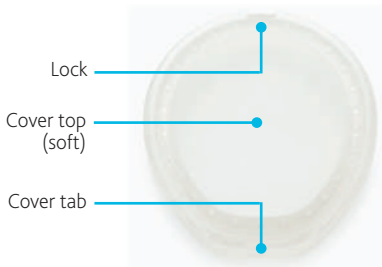
### Aqua+ base

## Aqua+ for Kanso 2 instructions

1. If you use a Kanso 2 Safety Line, **remove** it from your sound processor.

Place your thumbnail in the slot at the base and **lift**.

2. **Remove** the Aqua+ cover.

Put your thumb under the cover tab and push upwards.

### CAUTION

Don't damage the cover with your fingernails.

3. **Check** both halves for damage (e.g. scratches, tears) or contamination (e.g. hair, sunscreen). If it is damaged, use a new Aqua+.

### CAUTION

Anything that affects the seal may cause water to leak in and damage your sound processor.

4. Place your sound processor in the Aqua+ base with the **Cochlear logo, light and microphones facing up.**

#### TIPS

- Your Aqua+ is shaped so your processor only fits one way.
- If you use a Cochlear SoftWear™ pad, you can leave it on the processor.

5. **Align** the lock on the cover and base and fit them together.

After the lock parts are fitted, **lower** the other end of the cover onto the base.

#### TIP

Make sure you **don't trap too much air** in the cover. This could affect sound quality and comfort.

6. **Press firmly** around the edges until the base is flush with the cover.

**CAUTION**

Make sure your Aqua+ is properly sealed, so water doesn't leak in and damage your processor.

7. If you're worried about losing Aqua+, use a Nucleus Safety Line to **attach** it to your clothing.

See *Safety Line instructions* on page 10.

8. **Place** the Aqua+ on your implant as normal, making sure the Cochlear logo, light and microphone ports are facing up.

9. To keep Aqua+ on your implant, use a retention aid such as **goggles**.

**NOTE**

Make sure your sound processor is still in the correct position on your implant.

10. Finally, do the **Aqua+ for Kanso 2 Checklist** to make sure you're ready!

Aqua+ tightly sealed

Safety Line attached if required

Retention aid ready to use

**CAUTION**

Remember, you can't go deeper than 3 metres.

## Safety Line instructions

### WARNING

Retention lines longer than the Safety Line (short length) are not recommended for use by children under 3 years as they may present a risk of strangulation.

1. **Thread** the loop end of the line through the hole in the Aqua+.

2. Create a **knot** by passing the clip through the line loop. Pull it tight.

3. **Lift** the lever to open the clip.


4. Place the clip on your clothing and **press down** on the lever to close the clip.

## Care and storage

1. **Rinse** Aqua+ with clean water and dry with a soft cloth.
2. **Open** the Aqua+ cover and remove your sound processor.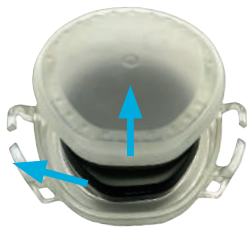

3. Check that the Aqua+ is **thoroughly dry**.
4. Put the two halves together and **test** the seal by pressing down gently on the cover.

If any air leaks out you should not use it again.

5. If the seal is ok, **separate the two halves again** and place your Aqua+ in your storage case or the Kanso 2 Activity Kit's waterproof case until you want to use it again.

## Cautions

- Avoid pressing with your fingernails as this can damage Aqua+.
- If your Aqua+ appears damaged or worn, replace it with a new one.
- If your sound processor does get wet, dry it with a soft dry cloth and place it in your Home Charger for at least eight hours.
- If you use a Softwear pad and experience intermittent sound from your processor, try removing the Softwear pad.

## Warnings

- If your sound processor becomes unusually warm, remove it immediately and seek advice from your clinician.  
Parents/caregivers: If your recipient indicates discomfort, check if their sound processor is unusually warm.
- Small parts (e.g. socket cover, safety line) may be a choking or strangulation hazard.
- If you experience tightness or pain at the implant site, or develop significant skin irritation, stop using your sound processor and contact your clinician.
- If you experience tightness or pain from your retention aid (e.g. goggles), either loosen it or replace it with a larger retention aid.
- If the magnet is too strong, pressure sores may develop at the implant site. If this happens, or if you experience any discomfort in this area, contact your clinician.
- Store spare magnets safely and away from cards that may have a magnetic strip (e.g. credit cards, bus tickets, etc).
- No modification of this equipment is allowed.
- Use only as intended.
- Report unexpected problems to your clinician.

## Other information

### Environmental conditions

CONDITION	MINIMUM	MAXIMUM
Storage and transport temperature	-10°C (14°F)	+55°C (131°F)
Storage and transport humidity	0% RH	90% RH
Operating temperature	+5°C (41°F)	+40°C (104°F)
Operating relative humidity	0% RH	90% RH
Operating pressure	700 hPa	1060 hPa

### Warranty

The use of Aqua+ for Kanso 2 will not impact your Nucleus Global Limited Warranty. Your Nucleus Global Limited Warranty will not be void in circumstances where Aqua+ for Kanso 2 is used in water with Kanso 2 Sound Processors only, in accordance with the Aqua+ for Kanso 2 instructions.

### Legal statement

The statements made in this guide are believed to be true and correct as of the date of publication. However, specifications are subject to change without notice.

© Cochlear Limited 2020

## Labelling symbols

The following symbols may appear on Aqua+ components and/or packaging.

Refer to instruction manual

Specific warnings or precautions associated with the device, which are not otherwise found on the label

Manufacturer

Authorised representative in the European Community

Catalogue number

Serial number

Batch code

Date of manufacture

Temperature limits

CE registration mark

**Rx Only**

By prescription

Recyclable material

Dispose of electrical components in accordance with your local regulations

Ingress Protection Rating:

**IP68**

- Protected against failure from dust penetration
- Protected against failure from continuous immersion in water up to 3 metres deep for up to 2 hours

## Serious incidents

Whilst serious incidents in relation to medical devices are rare, it is acknowledged that incidents may happen. As an organisation, Cochlear recognises the potential for harm and will respond to any reported serious incident.

### What is a serious incident?

A 'serious incident' means any event that directly or indirectly has caused or could have caused an unexpected or unwanted event including any of the following:

- The death of a patient, user or other person,
- The temporary or permanent serious deterioration of a patient's, user's or other person's state of health,
- A serious public health threat.

### Reporting a serious incident

There is no definitive list of events/incidents that constitute a serious incident, however all serious incidents should be reported to:

- your local Cochlear office  
[www.cochlear.com/intl/contact/global-offices](http://www.cochlear.com/intl/contact/global-offices)
- your National Competent Authority [http://ec.europa.eu/growth/sectors/medical-devices/contacts\\_en](http://ec.europa.eu/growth/sectors/medical-devices/contacts_en)

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**Cochlear Canada Inc** 2500-120 Adelaide Street West, Toronto, ON M5H 1T1, Canada  
Tel: +1 416 972 5082 Fax: +1 416 972 5083

**Cochlear AG** EMEA Headquarters, Peter Merian-Weg 4, 4052 Basel, Switzerland  
Tel: +41 61 205 8204 Fax: +41 61 205 8205

**Cochlear Europe Ltd** 6 Dashwood Lang Road, Bourne Business Park, Addlestone, Surrey KT15 2HJ, United Kingdom  
Tel: +44 1932 26 3400 Fax: +44 1932 26 3426

**Cochlear Benelux NV** Schaliënhoedreef 20 i, B-2800 Mechelen, Belgium  
Tel: +32 15 79 55 11 Fax: +32 15 79 55 70

**Cochlear France S.A.S.** 135 Route de Saint-Simon, 31035 Toulouse, France  
Tel: +33 5 34 63 85 85 (International) or 0805 200 016 (National) Fax: +33 5 34 63 85 80

**Cochlear Italia S.r.l.** Via Larga 33, 40138 Bologna, Italy  
Tel: +39 051 601 53 11 Fax: +39 051 39 20 62

**Cochlear Nordic AB** Konstruktionsvägen 14, 435 33 Mölnlycke, Sweden  
Tel: +46 31 335 14 61 Fax: +46 31 335 14 60

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**Cochlear (HK) Limited** Room 1404-1406, 14/F, Leighton Centre, 77 Leighton Road, Causeway Bay, Hong Kong  
Tel: +852 2530 5773 Fax: +852 2530 5183

**Cochlear Korea Ltd** 1st floor, Cheongwon Building 33, Teheran-ro 8 gil, Gangnam-gu, Seoul, Korea  
Tel: +82 2 533 4450 Fax: +82 2 533 8408

**Cochlear Medical Device (Beijing) Co., Ltd**  
Unit 2608-2617, 26th Floor, No.9 Building, No.91 Jianguo Road, Chaoyang District, Beijing 100022, P.R. China  
Tel: +86 10 5909 7800 Fax: +86 10 5909 7900

**Cochlear Medical Device Company India Pvt. Ltd.**  
Ground Floor, Platina Building, Plot No C-59, G-Block, Bandra Kurla Complex, Bandra (E), Mumbai – 400 051, India  
Tel: +91 22 6112 1111 Fax: +91 22 6112 1100

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**Cochlear Middle East FZ-LLC**  
Dubai Healthcare City, Al Razi Building 64, Block A, Ground Floor, Offices IR1 and IR2, Dubai, United Arab Emirates  
Tel: +971 4 818 4400 Fax: +971 4 361 8925

**Cochlear Latinoamérica S.A.**  
International Business Park, Building 3835, Office 403, Panama Pacifico, Panama  
Tel: +507 830 6220 Fax: +507 830 6218

**Cochlear NZ Limited**  
Level 4, Takapuna Towers, 19-21 Como St, Takapuna, Auckland 0622, New Zealand  
Tel: + 64 9 914 1983 Fax: 0800 886 036

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ACE, Advance Off-Stylet, AOS, AutoNRT, Autosensitivity, Beam, Bring Back the Beat, Button, Carina, Cochlear, 科利耳, コクレア, 코클리어, Cochlear SoftWear, Codacs, Contour, Contour Advance, Custom Sound, ESPrit, Freedom, Hear now. And always, Hugfit, Hybrid, Invisible Hearing, Kanso, MET, MicroDrive, MP3000, myCochlear, mySmartSound, NRT, Nucleus, Outcome Focused Fitting, Off-Stylet, Slimline, SmartSound, Softip, SPrint, True Wireless, the elliptical logo, and Whisper are either trademarks or registered trademarks of Cochlear Limited. Ardium, Baha, Baha SoftWear, BCdrive, DermaLock, EveryWear, SoundArc, Vistafix, and WindShield are either trademarks or registered trademarks of Cochlear Bone Anchored Solutions AB.

© Cochlear Limited 2020

D1708073 ISS1 APR20

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**Cochlear Tıbbi Cihazlar ve Sağlık Hizmetleri Ltd. Şti.**